




EPS-30/ EPS-30LP

Endoscopy Positioning System

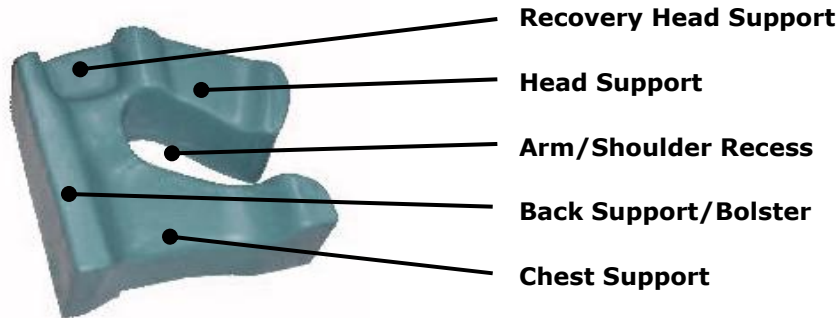

Instructions For Use

1. For ease of use, position patient in desired position prior to administration of sedation or general anesthesia
2. The EPS-30 is designed to accommodate a variety of positioning techniques including left lateral recumbent, prone, and supine
3. For best results, ensure that the patient is positioned upward (superior) on the pillow sufficiently to allow the top of their shoulder to touch the head support
4. Refer to supplemental pictures for additional positioning instructions
5. The EPS-30 is designed to provide function and support throughout the procedure and during recovery. Use care when removing the EPS-30 to avoid unnecessary strain on the patient or positioner.
6. Discontinue use immediately if patient reports or demonstrates cervical or upper extremity discomfort.

Care and Cleaning

1. The EPS-30 is designed for multiple uses with proper care and cleaning
2. Use care when removing from patient to avoid tearing
3. Clean after each use with soap and water or mild sanitizer
4. Do not submerge
5. Do not autoclave
6. Do not puncture or cut the protective covering

For questions or order information, contact:

Order #: 69151/69151LP



Vortek Surgical, LLC
Indianapolis, IN
(888) 867-8351
www.vorteksurgical.com

Reusable

NONSTERILE

